




Status™


# COVID-19/Flu A&B

A Rapid Immunoassay for the Simultaneous  
Direct Detection and Differential Diagnosis of SARS-CoV-2,  
Influenza Type A and Type B Antigen from anterior nasal and  
nasopharyngeal swab specimens

Infections with these viruses may present similar symptoms.  
Can you tell them apart? **We Can.**

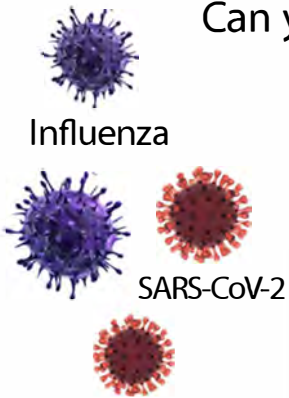

Learn More

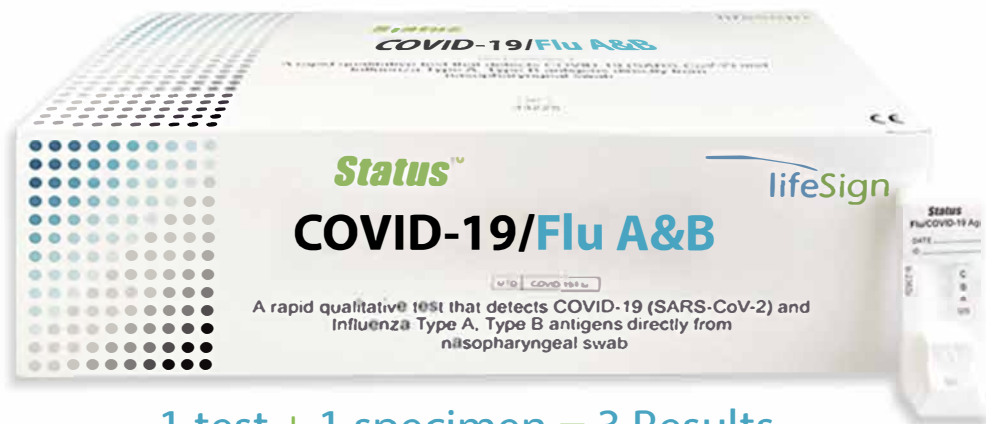


1 test + 1 specimen = 3 Results

- **COVID-19 - Anterior Nasal - Sensitivity 93.8%, Specificity 100%**  
**Nasopharyngeal - Sensitivity 93.1%, Specificity 100%**
- **Flu A - Sensitivity 91.4%, Specificity 95.7%**
- **Flu B - Sensitivity 87.6%, Specificity 95.9%**
- **FDA Emergency Use Authorization (EUA)**
- **Visually read in 15 minutes**
- **Flocked swabs for superior specimen collection and patient comfort**



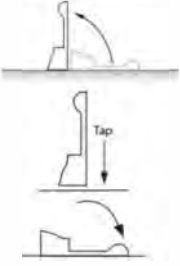
ORDERING INFORMATION		
PRODUCT	# OF TESTS	ITEM #
Status™ Covid-19/Flu A&B	25	33225

Proudly made in the USA

For more information or to place an order, contact:

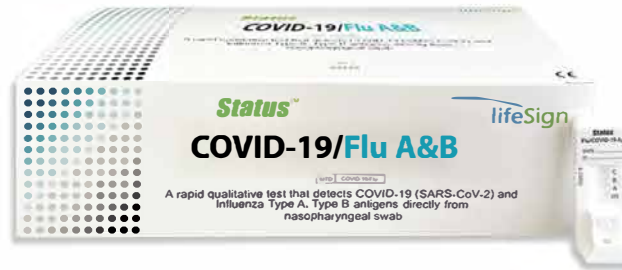

## Easy 3 Step Procedure
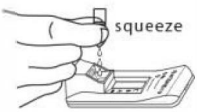
<b>1</b>		squeeze Tear the tab off the Extraction Reagent Capsule and dispense entire contents into the Extraction Well.
<b>2</b>		Insert the specimen swab in the Swab Stand. • <b>Rotate</b> swab 3 times to mix the specimen. • <b>Let stand</b> 1 minute. • <b>Rotate</b> swab 3 times again and discard the swab.
<b>3</b>		Raise the device upright and <b>let stand</b> 1–2 seconds.  Gently <b>tap</b> the device to ensure the liquid flows into the hole.  Lay the device back down.
<b>Set a timer for 15 minutes.</b>		
<b>4</b>	<b>Read test results at 15 minutes.</b> <b>NOTE: False positive or false negative results can occur if the test is not read between 15 and 20 minutes.</b>	

- Refer to our website at [www.lifesignmed.com](http://www.lifesignmed.com) for complete test procedure details, training video and package insert.
- Status™ COVID-19/FLU A&B is an antigen test for use with anterior nasal and nasopharyngeal swab specimens.
- Status™ COVID-19/FLU A&B test has been granted emergency use authorization (EUA) by the FDA for use at the point-of-care setting. During the duration of the emergency this test can be performed in a patient care setting that is operating under a CLIA Waiver, Certificate of Compliance or Certificate of Accreditation.

 Proudly made in the USA

For more information or to place an order, contact:

  
[AtlanticMedicalSolutions.com](http://AtlanticMedicalSolutions.com)  
 704-561-0004